




Capio™ SLIM
Suture Capturing Device



Capio™ SLIM

Suture Capturing Device

The Capio SLIM Suture Capturing device features a reduced profile to minimize the space required within the surgical field, easy suture loading and a funnel shaped catch, designed for consistent ease of use.

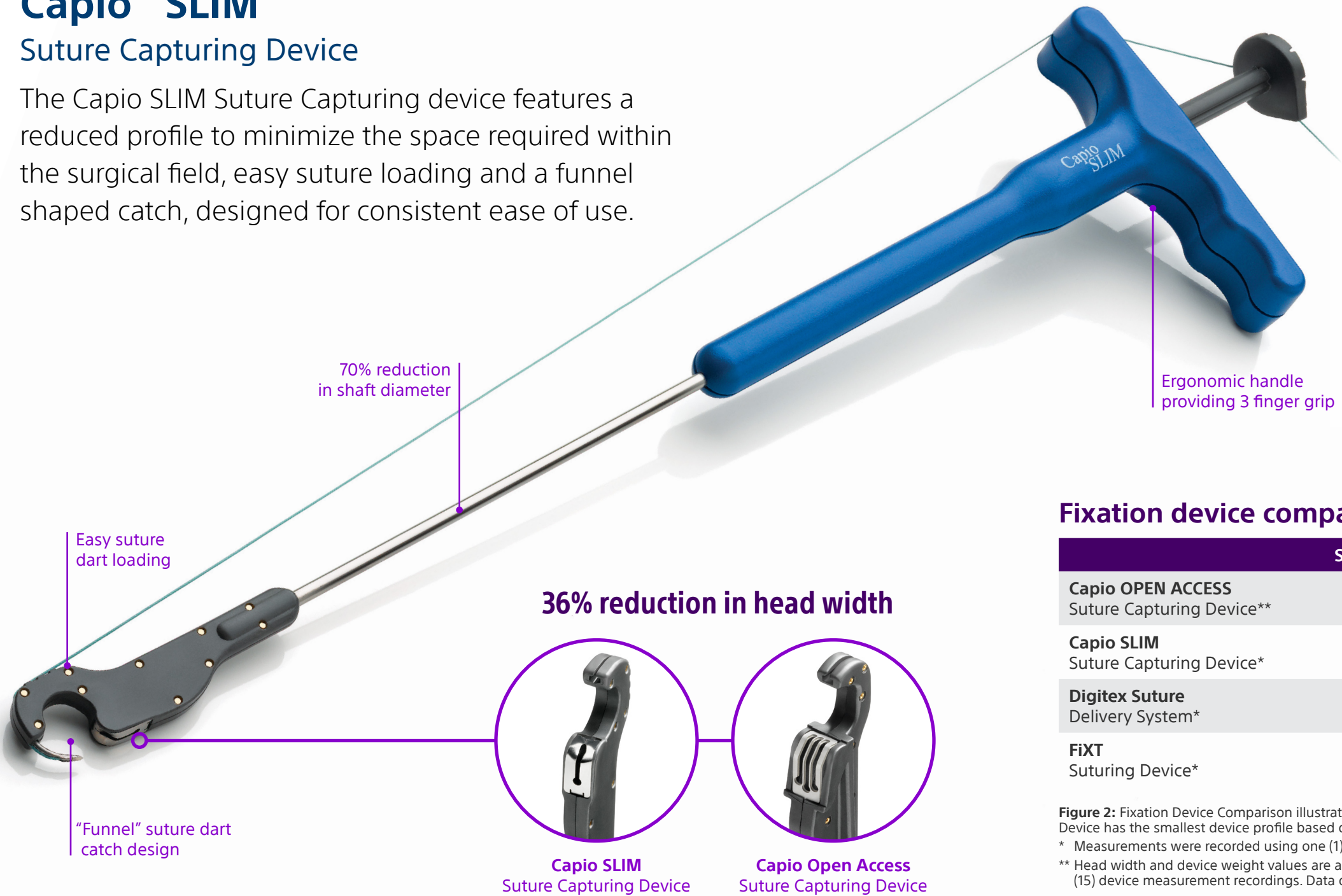

Comparison of suture capturing device heads

Building on the history and reducing the profile, the Capio SLIM Suture Capturing Device features one of the smallest device profiles and suture dart carrier diameter while keeping the same bite depth as the current Capio Open Access Suturing Device.

Capio™ SLIM
Suture Capturing Device

Digitex™
Suture Delivery System

FiXT™
Suturing Device

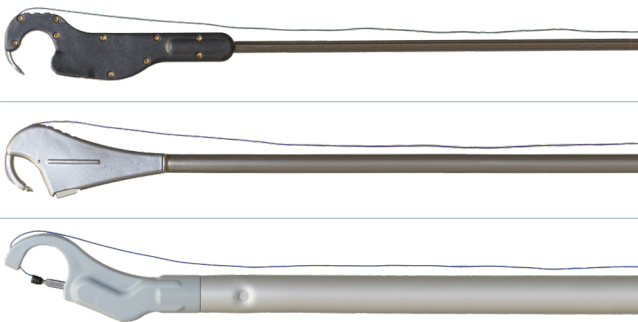


Figure 1: Comparison of Device Heads illustrates the difference in carrier diameter amongst these fixation devices. The Capio SLIM Suture Capturing Device carrier diameter is 45% less than the FiXT Suturing Device , 43% less than the Digitex Suture Delivery System.

Fixation device comparison

	Shaft Diameter	Head Width	Device Weight	Carrier Diameter
Capio OPEN ACCESS Suture Capturing Device**	10.1 mm	9.8 mm	45.4 g	1.2 mm
Capio SLIM Suture Capturing Device*	3.0 mm	6.3 mm	38.2 g	1.2 mm
Digitex Suture Delivery System*	9.5 mm	9.6 mm	133.3 g	2.1 mm
FiXT Suturing Device*	4.8 mm	6.9 mm	121.1 g	2.2 mm

Figure 2: Fixation Device Comparison illustrates the device profile differences amongst these fixation devices. The Capio SLIM Suture Capturing Device has the smallest device profile based on the following measurements: shaft diameter, head width and device weight.

* Measurements were recorded using one (1) of each device. Data on file at Boston Scientific.

** Head width and device weight values are an average of five (5) device measurement recordings. Shaft diameter values are an average of fifteen (15) device measurement recordings. Data on file at Boston Scientific.

The evolution continues with the Capio SLIM Suture Capturing Device

Since 1995, the Capio Device has provided consistent suture placement in difficult to access pelvic floor locations.

1995

The Capio Suture Capturing Device debuts, revolutionizing the field of urogynecology.

1999

Capio Device may be used with other biological graft materials offering another treatment option

2011

The Capio Device remains “the foundation” of Pelvic Floor Reconstruction.

2012

Capio SLIM: the evolution continues.

Capio SLIM Suture Capturing Device

Order Number	Description	Unit
M0068318250	Capio Slim Suture Capturing Device	bx 1
M0068318261	Capio Slim Suture Capturing Device	bx 5

Capio Sutures

Order Number	Description	Size	Unit
M0068332341	Non-absorbable, polypropylene monofilament, double armed with two TC tapercut needles (darts), 48"	0	bx 12
M0068332351	Non-absorbable, polypropylene monofilament, double armed with TC tapercut needle (dart) and a T 26mm 1/2 circle taper needle, 36"	0	bx 12
M0068332231	Non-absorbable, coated braided polyester, double armed with two TC tapercut needles (darts), 48"	0	bx 12
M0068332241	Non-absorbable, coated braided polyester, double armed with TC tapercut needle (dart) and a T 26mm 1/2 circle taper needle, 36"	0	bx 12
M0068331371	Monodek™ Absorbable, monofilament PDO, double armed with TC tapercut needle (dart) and a T 26mm 1/2 circle taper needle, 48"	0	bx 12
M0068332452	Bondek™ Plus Suture, absorbable, coated braided PGA, double armed with two TC tapercut needles (darts), 48"	0	bx 12

Your pelvic floor reconstructive tool to meet a variety of your algorithm needs:

- Native tissue repair
- Biologic graft augmentation
- And more...

Indications: The Capio SLIM Device is intended for use in general suturing applications during urogynecologic surgery to assist in the placement of suture material in tissues at the operative site. It is also intended for use as an aid in the insertion, placement, fixation, or anchoring of surgical grafts during urogynecologic procedures. The Capio SLIM Device is to be used with sensory and/or direct visual control. **Contraindications:** The use of the Capio SLIM Device is contraindicated for placing sutures into or through bone. **Warnings:** Punctures or lacerations of vessels, nerves, bladder, urethra or bowel may occur during placement and may require surgical repair. Once the suturing action has begun, do not turn, rotate or torque the Capio SLIM Device. To do so, may result in injury. **Precautions:** Use only size 0 Capio Sutures with the Capio SLIM Device. The physician is advised to consult the medical literature regarding techniques, complications and hazards associated with the intended procedures. The Capio SLIM Device is not intended to be used to manipulate tissue, organ or bone. Do not bend or alter the device as this may result in mechanical damage to the device. **Potential Adverse Events:** Potential complications associated with the Capio SLIM Device include, but are not limited to: allergic reaction, bleeding, dyspareunia, hematoma, infection, inflammation, injury to internal organs/tissue including perforation and occlusion, injury to vessels and nerves, pain, and unretrieved device fragment.

CAUTION: Federal Law (USA) restricts these devices to sale by or on the order of a physician. This product is intended for use only by clinicians with adequate training and experience in urogynecologic procedures. Refer to package insert provided with these products for complete Indications for Use, Contraindications, Warnings, Precautions, Adverse Events, and Instructions prior to using these products.

All images are the property of Boston Scientific. All trademarks are the property of their respective owners.



Boston Scientific Corporation
300 Boston Scientific Way
Marlborough, MA 01752-1234
www.bostonscientific.com

© 2025 Boston Scientific Corporation
or its affiliates. All rights reserved.

WH-125306-AE MAY 2019