CelebrityTM Cytology Brush

Endoscopic Cytology Brush

en	Instructions for Use	2
es	Instrucciones de uso	5
fr	Mode d'emploi	8
de	Gebrauchsanweisung	11
it	Istruzioni per l'uso	15
nl	Instructies voor gebruik	19
pt-EU	Instruções de Utilização	22

51634533-01

2024-05

BSC (MB IFU Template 3 x 9 Global, 92310029R), IFU, MB, Celebrity Cytology Brush, Global 7, 51634533-01B

Cellebrity™ Cytology Brush

Endoscopic Cytology Brush

Rx ONLY

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

REUSE WARNING

For single use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

DEVICE DESCRIPTION

Contents

The package includes:

One (1) Cellebrity Endoscopic Cytology Brush

Operating Principle

Cellebrity Endoscopic Cytology Brush is indicated for acquiring tissue samples used for the diagnosis of suspected pathology in the airway tree. The device is used with the aid of a bronchoscope.

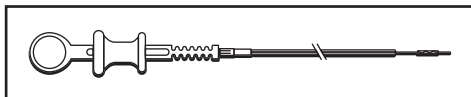
Figure 1 – Cellebrity Cytology Diagram

Cellebrity Endoscopic Cytology Brushes are twisted wires ensnaring fibers (polymer or metal) to shape the form of a cylindrical brush. The bristle outer diameter of the cytology brush range in sizes (1.0 mm, 1.5 mm and 1.9 mm). This assembly is then inserted into a PTFE sheath. A hand-operated brush extension handle is attached to the proximal end of the assembly. The brush may be extended from the sheath, by means of the handle, to capture a cellular sample onto the bristles. The brush is then retracted into the sheath to protect the sample.

Materials

Contains Cobalt: CAS No. 7440-48-4; EN No. 231-158-0.

Defined as a 1B carcinogen and reproductive toxicant according to the European Commission in a concentration above 0.1% weight by weight.

Note: Current scientific evidence supports that metal alloys containing cobalt used in medical devices, do not cause an increased risk of cancer or adverse reproductive effects.

Compatibility

The Cellebrity Endoscopic Cytology Brush is designed for use with a bronchoscope working channel that has a diameter of at least 2.0 mm. Any use of this device, other than those indicated in these instructions, is not recommended.

User Information

The Cellebriety Endoscopic Cytology Brush should only be used by or under the supervision of physicians thoroughly trained in bronchoscopic procedures. A thorough understanding of the technical principles, clinical applications, and risks associated with bronchoscopic procedures is necessary before using this device.

INTENDED USE/INDICATIONS FOR USE

The cytology brush is indicated for acquiring tissue samples during bronchoscopy for the diagnosis of suspected pathology in the airway tree.

Clinical Benefit Statement

The clinical benefit of the Cellebriety Endoscopic Cytology Brush is to allow for endoscopic collection of tissue to aid in the diagnosis of pathology in the airway tree.

CONTRAINDICATIONS

None.

WARNINGS

- In order to avoid patient injuries, the device should be used with caution and only after careful consideration in patients with coagulopathy and in those whose general medical condition and degree of respiratory failure may impact the patient’s tolerance of bronchoscopy (rigid or flexible) and/or the manipulation required to perform the procedure.
- Do not use excessive force if resistance is met when using the device, as it may lead to patient injuries.
- Ensure adequate visualization is established during the use of the brush to avoid brushing blindly which may lead to patient injuries.

ADVERSE EVENTS

Potential complications include, but may not be limited to:

- Aspiration
- Bronchospasm
- Hemorrhage
- Infection/Pneumonia
- Obstruction of airway
- Pain
- Perforation
- Pneumothorax
- Respiratory Distress/Insufficiency
- Respiratory Failure
- Tissue Damage

HOW SUPPLIED

The Cellebriety Endoscopic Cytology Brush is supplied sterile using an ethylene oxide (EO) process. The packaging and device should be inspected prior to use.

Device Details

Do not use if package is damaged or unintentionally opened before use.

Do not use if labeling is incomplete or illegible.

Handling and Storage

This product has no special handling or storage requirements. Check device label for expiration date.

OPERATIONAL INSTRUCTIONS

Procedure

1. Remove the Celebrity Endoscopic Cytology Brush from its package and inspect for any damage.
2. With the cytology brush retracted, insert device into accessory channel of bronchoscope.
3. Position the Celebrity Endoscopic Cytology Brush at the desired brushing site using direct visualization or fluoroscopic guidance.
4. Advance brush out of sheath by inserting thumb into thumb ring and opening the device to extend the brush.
5. Brush the site several times. After the brushing is complete, retract the brush into the catheter by closing the device to protect sample.
6. The catheter can then be removed from the bronchoscope.
7. Prepare specimen for examination per institutional guidelines.

Disposal

To minimize the risk of infection or microbial hazards after use, dispose device and packaging as follows:

After use, device may contain biohazardous substances. The device and packaging should be treated and disposed of as biohazardous waste or have them treated and disposed of in accordance with any applicable hospital, administrative, and/or local government regulations. Use of a biohazardous container with biological hazard symbol is recommended. Untreated biohazardous waste should not be disposed of in the municipal waste system.

This device contains a sharps hazard. Take precautions to ensure that sharps are handled properly. Dispose of all sharps directly into a sharps disposal container labeled with a biological hazard symbol. Sharps waste should be safely disposed of using available sharps waste channels in accordance with hospital, administrative, and/or local government policy.

Post-Procedure

Any serious incident that occurs in relation to this device should be reported to the manufacturer and relevant local regulatory authority.

PATIENT COUNSELING INFORMATION

Brief the patient on any relevant post-procedure instructions, contraindications, warnings, precautions and/or adverse events found within this Instructions for Use (IFU), pertaining to the patient.

WARRANTY

For device warranty information, visit
(www.bostonscientific.com/warranty).

The following are trademarks of Boston Scientific Corporation or its affiliates:
Celebrity.

All other trademarks are the property of their respective owners.

SYMBOL DEFINITIONS

Commonly used medical device symbols that appear on the labeling are defined at www.bostonscientific.com/SymbolsGlossary.

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or its affiliates. All rights reserved.
Boston Scientific (Australia) Pty Ltd
Boston Scientific Limited
© 2024 Boston Scientific Corporation